Platelet PGD® Microfuge Tubes and Disposable Pipettes

VERAX BIOMEDICAL
INCORPORATED

A. INTENDED USE

The Platelet PGD Microfuge Tubes and Disposable Pipettes are only for use with the Platelet PGD Test.

B. SUMMARY AND EXPLANATION

Platelet PGD Microfuge Tubes and Disposable Pipettes are single use, disposable accessories for use with the Platelet PGD Test. They are used in the processing of platelet samples prior to addition to the Platelet PGD Test Device for detection of aerobic and anaerobic Gram-positive and Gram-negative bacteria and in the processing of Platelet PGD Controls.

C. PRINCIPLE OF THE PROCEDURE

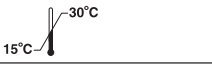
Platelet PGD Microfuge Tubes and Disposable Pipettes are specifically for use with the Platelet PGD Test. Similar laboratory disposables should not be substituted.

D. REAGENTS AND MATERIALS

Materials Provided

Microfuge Tubes and Disposable Pipettes for the Platelet PGD Test **REF** P100D-CE

Includes the following:

	100 Tests
Disposable Pipettes	100 each
Microfuge Tubes	100 each

Materials Required

Platelet PGD Test **REF** P20CE 20 Tests
REF P100CE 100 Tests

Materials Required But Not Provided - Refer to the Platelet PGD Test Instructions for Use (IFU).

Platelet PGD® Microfuge Tubes and Disposable Pipettes

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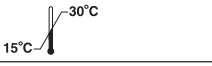
For *In Vitro* Diagnostic Use

Warnings

- Read the Platelet PGD Test and Platelet PGD Controls IFU completely before using the product. Follow the instructions carefully. Not doing so may result in inaccurate test results.
- The Platelet PGD Microfuge Tubes and Disposable Pipettes have been validated for use with the Platelet PGD Test only. Do not substitute similar laboratory disposables.

Handling Precautions

Handle and perform test properly: Do not re-use Microfuge Tubes or Disposable Pipettes.

F. STORAGE INSTRUCTIONS

Store Platelet PGD Microfuge Tubes and Disposable Pipettes at 15 - 30 °C.

For the following sections - Refer to the Platelet PGD Test IFU and the Platelet PGD Controls IFU.

G. SPECIMEN COLLECTION AND PREPARATION FOR ANALYSIS

H. TEST PROCEDURE

I. LIMITATIONS

J. REFERENCES

Verax Biomedical and PGD are trademarks of Verax Biomedical Incorporated.

For Customer Service:

For additional product information, please contact your local Fenwal customer service organization.

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© 2009 Verax Biomedical Incorporated
Fenwal Inc.
Canada: Source Medical Corporation